

Introducing SoClean.® 2

My goal is to make CPAP therapy a
healthier and easier experience.

The SoClean CPAP sanitizing machine uses safe, natural, activated oxygen to thoroughly and effortlessly sanitize your entire CPAP system by eliminating mold, bacteria, and viruses.



CPAP equipment, mask, or hose not included.

#1
As Rated By
CPAP USERS



A safe, easy & natural way to thoroughly sanitize your CPAP equipment daily.



The SoClean 2 is an automated CPAP equipment sanitizer. Like the original SoClean, SoClean 2 sanitizes CPAP equipment—your mask, hose, and reservoir without taking your equipment apart. Full and complete sanitizing is accomplished without the use of messy chemicals or water.

Features

- No equipment disassembly
- Small footprint
- New sleek design
- Front clock and display
- Status indicator lights
- Right or left hose insertion option
- Easy filter cartridge system
- Automatic safety shut-off



The inside chamber was designed specifically so that a wide range of mask types can be used within the SoClean unit, from nasal pillow masks to full CPAP masks. SoClean uses a universal fitting to work with an array of CPAP models.


Benefits

- Protect your health
- Destroys 99.9% of bacteria, viruses & mold
- Supports a wide range of mask types
- Completely automated
- Time-saving operation
- No water or chemicals



How does the SoClean work?

The SoClean uses Activated Oxygen — A powerful sanitizing agent. Activated Oxygen is created when a third atom of oxygen combines with the two-atom oxygen molecule that makes up the air we breathe. This new molecule has the ability to destroy organic matter such as mold, viruses, and bacteria. This active ingredient pumps through your CPAP equipment, sanitizing along the way. A special filter then converts the activated oxygen back into the air we breathe.

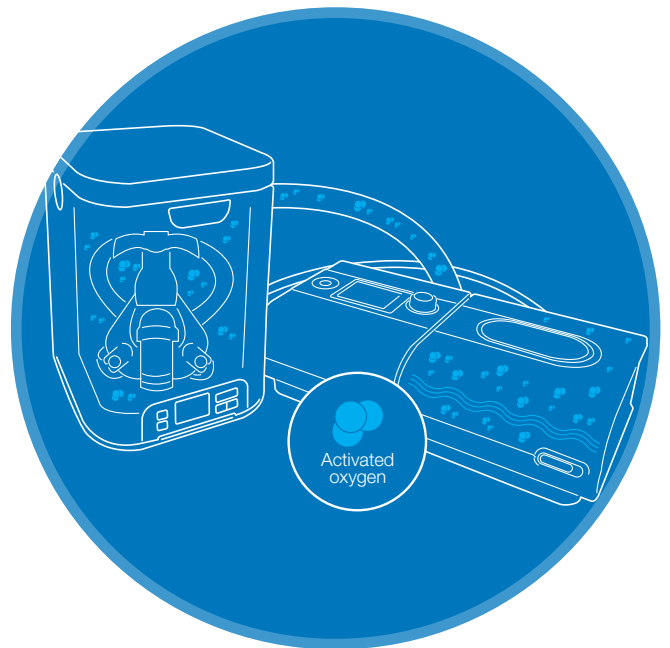

SoClean uses the same sanitizing process also found in:



Water purification



Produce storage



Hotel housekeeping



Hospital sanitizing